Mag-Mat

(PS4116)

EN

DIRECTIONS FOR USE

The following information should be read before using this device.

Indications:

The Purple Surgical Mag-Mat minimises the risk of accidental injuries by providing an alternative to hand to hand transfer of surgical instruments. The Mag-Mat is a secure, non-slip magnetic mat that is conveniently situated close to the operative site.

Contraindications:

The Mag-Mat should not be used on a patient with a pacemaker, or similar device, without first checking compatibility with the device manufacturer.

Device Description:

The Purple Surgical Mag-Mat is constructed of high grade, non-porous rubber coated cloth and round ceramic magnets.

Directions For Use:

- The Mag-Mat should be placed close to the surgical site.
- Instruments should be placed on, and retrieved from, the mat thus preventing the necessity of hand to hand transfer.
- The Mag-Mat may be secured to the patient drape by means of a ball and socket clamp being passed through the holes in the corners of the mat.

Care:

AUTOMATED CLEANING

It is recommended that the drape is decontaminated using an automated method of cleaning / disinfection as prescribed in HTM2030, however the product life may be reduced if care is not taken during reprocessing. Care should be taken to prevent damage to the Mag-Mat from contact with sharp objects and ensuring that no folding of the Mag-Mat occurs during processing. Detergent compatibility should be verified with the detergent manufacturer.

MANUAL CLEANING

- Fill a bowl or sink with the indicated dose of mild detergent. Wipe the Mag-Mat to dislodge and remove all visible soil, taking care to ensure the item remains under the surface of the water at all times to prevent the creation of aerosols.
- Remove the item from the sink and drain any excess detergent prior to placing in the second sink to rinse in clean water.

- Rinse the item thoroughly with clean water or water jet gun under the surface of the water.
- Remove and drain the item before drying with lint-free material.
- Complete any necessary documentation to record the item being processed and the method and solutions employed.

Precautions/Warnings

Use of the following substances may perforate the Mag-Mat:

- Oil, ointment and oil containing soaps (green soap)
- Hydrocarbon solvents: ether, acetone, benzene, freon, alcohol or oxidizing agents.
- Sulphur, acetic and hydrochloric acids, lye and sodium hypochlorite (bleach).
- Phenol and hexachlorophene products.
- Quaternary compounds and benzalkonium chloride.

Liquid chemical sterilization is not recommended for this product.

Sterilization:

- Porous load steam sterilization is the recommended method of sterilization.
- If the Mag-Mat is to be sterilized using a porous load steam sterilizer it should be packaged in accordance with the relevant national / international standard.
- The Mag-Mat is suitable for sterilization in a porous load sterilizer operated, maintained and validated in accordance with HTM2010.
- The Mag-Mat can withstand autoclave temperature up to 137°C.
- In order to prolong the life of the Mag-Mat it should be sterilized positioned flat on the loading carriage and not subjected to heavy objects being placed on top.

How Supplied

The Purple Surgical Mag-Mat is supplied non-sterile and is a re-usable product.

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Purple Surgical Holdings Limited
also known as:
Purple Surgical International Limited,
Purple Surgical UK Limited,
Purple Surgical Manufacturing Limited,
2 Chestnut House, Farm Close, Shenley, Herts, WD7 9AD, UK
T: +44 (0)1923 839333 F: +44 (0)1923 839444
E: mail@puplesurgical.com W: www.puplesurgical.com

Labelling and IFU Symbology

		
Manufacturer	Catalogue number	Batch code
		
Do not use if package is damaged	Consult instructions for use	Keep dry
		
Caution	Not Sterile	Keep away from sunlight